







How to use the <b>FM transmitter</b> and <b>auxillary cable</b> <i>instead of headphones</i> for Deaf/hard of Hearing Students	
<p>Unplug the headphone/microphone (in this picture you see a student unplugging a USB cord) from the chromebook</p>	
<p>Plug the auxiliary cord (in this picture it is a red cord) into the FM transmitter into the headphone jack</p>	
<p>Plug the auxiliary cord (in this picture the red cord) into the sound source (the computer's HEADPHONE jack).</p> 	
<p>The student may also desire to adjust the main volume for the sounds coming from the computer.</p>	
<p>Plug in the original headphone/microphone back into the chromebook</p>	
<p><b>Happy Listening</b> through you Hearing Aids or Cochlear Implant using your FM transmitter and the auxiliary cable!</p>	