

Clip-On Microphone Cheat Sheet

This Roger microphone system connects to a student's hearing aids or cochlear implants, allowing the student to hear the teacher's voice clearly, no matter how much background noise there is in the room or how far away he or she is from the teacher. For more information on why a microphone system is used and what it sounds like, scan the QR code.



Primary Microphone

This is the Roger Clip-on Microphone.

This microphone should be worn by whoever is providing instruction, and passed from one person to the next if a different teacher or paraprofessional takes over.

If the student is working with a partner on an assignment, the partner can wear the microphone.



Best worn on lanyard

It is best to wear the microphone on a lanyard because it will always be the right distance from the wearer's mouth. The lanyard has a magnetic clasp so it can be easily transferred to another teacher/speaker.



To turn it on: Press and hold the power button (~2 sec). You will see a green light come on the front of the microphone. It will flash every few seconds to tell you it is on.

To mute the microphone: Do a quick press on the power button. The indicator light will change to purple. To unmute, do another quick press of the power button.

To turn it off: Press and hold the power button (~2 sec). You will see the light turn red and fade out.

To listen to the computer/ipad/chromebook etc:

Instead of a student wearing headphones over her hearing aids (which doesn't sound very good) you can use the audio cable and the microphone to send the computer sound through her hearing aids.

Step 1: Plug the audio cable mini USB end into the mini USB port on the bottom of the microphone.

Step 2: Plug the other end of the cable into the headphone port on the computer/ipad/chromebook etc.



Mic plugged into ipad headphone port



Remember to charge the microphone every day after school

By Amanda Levy, Educational Audiologist



Charging the Microphone