

## Roger Equipment Cheat Sheet

*This Roger microphone system connects to a student's hearing aids or cochlear implants, allowing the student to hear the teacher's voice clearly, no matter how much background noise there is in the room or how far away he or she is from the teacher. For more information on why a microphone system is used and what it sounds like, scan the QR code.*



**This is the Roger Pen Microphone.**



This microphone should be worn by whoever is providing instruction, and passed from one person to the next if a different teacher or paraprofessional takes over.

If the student is working with a partner on an assignment, the partner can wear the microphone.

It is best to wear the microphone on a lanyard because it will always be the right distance from the wearer's mouth.

The lanyard has a magnetic clasp so it can be easily transferred to another teacher/speaker.

During group discussion, the microphone can be put flat in the center of a table to pick up the voices of all around the table.



*Best to wear on lanyard*



*Can be placed in center of table during small group work*

**To turn it on:** Press and hold the power button (~2 sec). You will see a green light come on the front of the microphone. It will flash every few seconds to tell you it is on.

**To mute the microphone:** Do a quick press on the power button. The indicator light will change to purple. To unmute, do another quick press of the power button.

**To turn it off:** Press and hold the power button (~2 sec). You will see the light turn red and fade out.

**Remember to charge the Roger Pen every day after school.**

Line up the arrows on the pen and charger port.



## COMPUTER/IPAD/CHROME BOOK LISTENING:

*Instead of a student wearing headphones over her hearing aids (which doesn't sound very good) you can use the audio cable and the microphone to send the computer sound through her hearing aids.*

**Step 1:** Plug the audio cable mini USB end into the mini USB port on the bottom of the microphone.

**Step 2:** Plug the other end of the cable into the headphone port on the computer/ipad/chromebook etc.



*Mic plugged into ipad headphone port*



*By Amanda Levy, Educational Audiologist*